




Model #: _____ Serial #: _____


Using Your Equipment:

Pendant Control (if electric/semi-electric bed)

- Technician will explain or demonstrate use of hand pendant on how to raise/lower head and/or foot of bed.

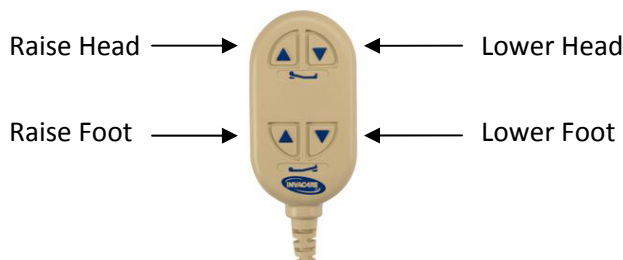

- Technician will explain the raise and lower option on foldaway crank (if fixed/variable height bed).

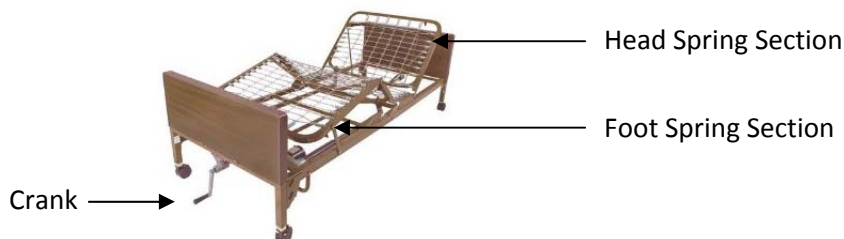

Siderails/Halfrails

- Technician will explain how to raise and lower side rails.

Trapeze Bar (if necessary)

- Technician will explain and demonstrate how to adjust grab bar and height of trapeze.
- Maximum weight capacity of trapeze in use is 250 lbs (patient assisted).

Maintenance/Inspection Checklist:

- ☐ Mattress - yearly check
- ☐ Check link springs to make sure all links are intact.
- ☐ Inspect head and foot boards for holes, cracks or sharp edges.
- ☐ Check for general wear or damage.
- ☐ Check pendant motor and power cords regularly for wear and proper function.
- ☐ Use of normal household cleaners is acceptable for regular cleaning.
- ☐ Contact us for all replacement parts or maintenance.

Safety Issues:

- Keep all moving parts, including the main frame, mattress deck and all drive shafts free of obstruction, i.e. blankets, sheets, tubing and other types of products that may get tangled around the bed.
- When transferring into or out of a homecare bed **always** engage wheel locks.
- Always test to make sure that the side rails are properly and securely in place before using the bed.
- Never use trapeze as a total individual weight support.
- Never operate unit if there is a damaged cord or plug. If unit is not running properly, contact your equipment provider immediately.
- If using bedside rails, be cautious of gaps between mattress and bedside rails where bodily harm may occur. Proper mattress size and/or raised mattress edges may help prevent gaps.

Troubleshooting:

If bed is not working properly, try the following solutions, or call our Service Department if unable to locate the problem at (888) 577-7331:

Cause

Solution

Bed is unplugged or connection is loose	—————>	Insert plug into wall outlet securely
Bed control is loose or disconnected from bed frame	—————>	Insert into outlet on bed securely
Power failure	—————>	Use emergency hand crank

Available Accessories (not all accessories are listed here – please ask staff about other items):



Over-Bed Table



Reacher



Bedpan



Bed Wedges



*Underpads
(disposable or
reusable)*



*Fitted Sheets
(beds require
'twin XL'
size sheets)*



Bed Alarm



Fall Mat