



## CADD PRIZM VIP PUMP INSTRUCTIONS

### *PCA MODE*

#### TO CHANGE VOLUME, RATE, BOLUS ( Patient Version)

Stop pump by pressing STOP/START.

\*\*\*Pump will need to be taken out of lock level before any changes to the program can be made. (Home Health nurses have the code to unlock the pump).

#### PUMP DISPLAY

#### KEY PRESS

Once pump is unlocked, proceed with the following:

***PCA	6210G ***
9V Battery	STOPPED
Press NEXT to advance	

Press “NEXT” until you get to the parameter that you wish to change.

Make changes as ordered, then press “ENTER” to accept the parameter change. When finished, press STOP/START.

Start the Pump?
Press Y or N

Press “Y”

\*\* Pump will now review the settings again and automatically lock to LL1. The following screen will appear, but then will go blank. Press NEXT at any time to bring back the display – and NEXT consecutively to check the parameters while pump is running.

***PCA	6210G ***
9V Battery	RUNNING
Res Vol	xxx.x ml
Press NEXT to advance	

\*\*\*green light will be blinking at the top of the pump while running and screen will go blank.

**PLEASE NOTE:** Pump will begin to beep at 1ml increments beginning at 5mls remaining to notify of “Low Reservoir Volume”. Both the yellow and green lights at the top of the pump will blink during this time. When infusion is finished and reservoir volume is zero, pump will begin with repeated Double Beeps. The screen will appear:

Reservoir Volume is Zero
Press NEXT to Continue